

3M™ Clear Custom Collar



Now you can clearly see the 3M advantage. . .

Since pets can destroy weeks of healing with just minutes of chewing, scratching or biting, protective collars are essential for the healing process. That's why we offer the 3M™ Clear Custom Collar – for pet comfort and customer satisfaction – all in one perfect fit.

The 3M protection collar idea originated from a practicing Minnesota veterinarian utilizing a


3M technology, the 3M Scotchmate™ Hook and Loop Fastener System. The collar was clinically developed and tested by veterinarians.

3M clear custom collars are available in five different sizes to accommodate a wide range of muzzle sizes; each collar can be custom fit to each individual animal's neck.

Features	Advantages	Benefits
<ul style="list-style-type: none"> • Clear 	<ul style="list-style-type: none"> • Visibility – easier for animal to see 	<ul style="list-style-type: none"> • Comfort for pet • Less disorientation • Customer satisfaction
<ul style="list-style-type: none"> • Customized fit 	<ul style="list-style-type: none"> • Custom fit to each individual animal • Fewer collar sizes to inventory 	<ul style="list-style-type: none"> • Better fit for animal • Sizing flexibility • Cost savings for clinic
<ul style="list-style-type: none"> • Easy application 	<ul style="list-style-type: none"> • Quick, easy application 	<ul style="list-style-type: none"> • Saves time, ease of use
<ul style="list-style-type: none"> • Padded neck edge 		<ul style="list-style-type: none"> • Improve animal comfort
<ul style="list-style-type: none"> • Scotchmate closure system 	<ul style="list-style-type: none"> • Quick, easy application 	<ul style="list-style-type: none"> • Easy to remove and reapply collar

3M™ Clear Custom Collar

1. Wrap the collar snugly around the pet's neck for correct sizing and optimum custom fit. While holding collar in place, mark desired position of the fastener with a pen (fig. 1).
2. Remove collar from animal. The Scotchmate fasteners remain mated together through the sizing procedure. For maximum adhesive bond to the collar, we recommend wiping the collar with isopropyl alcohol in the area that the fastener will be laid down. Allow alcohol to dry. Remove adhesive liner from the Scotchmate fastener to expose underlying adhesive. Avoid touching adhesive once liner has been removed (fig. 2).
3. Reform 3M collar into cone shape. Making sure the outer edges are even, lightly apply Scotchmate fastener to desired collar location. When you are sure of correct placement, firmly press and rub fastener to collar (fig. 3). After sizing collar to the animal, any excess collar material and neck padding overlap can be trimmed with a heavy scissors. Slits are provided at the neck edge for securing the collar to the pet's regular collar if necessary.
4. Apply collar to animal (fig. 4).

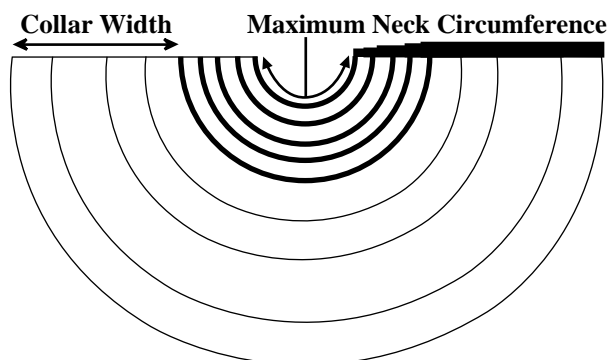

Cat. No.	Collar size	Maximum Neck Circumference	Collar width	Animal type	Collars per box
1450C	8	23.6 cm or 9.4 in	8 cm or 3.2 in.	Cats or small dogs with short muzzle	10
1450C	12	32.7 cm or 12.9 in.	12 cm or 4.7 in.	Small dogs with medium muzzle	10
1450C	17	39.3 cm or 15.4 in.	17 cm or 6.7 in.	Medium dogs with medium muzzle	10
1450C	23	53.7 cm or 21.2 in.	23 cm or 9.1 in.	Large dogs with medium muzzle	10
1450C	30	64.4 cm or 25.5 in.	30 cm or 11.8 in.	Extra large dogs with long muzzle	10
1450C		Assorted pack – 1 of each size collar			5

These products are available through your veterinary supply distributor. For questions on 3M products, please call 3M Customer Service at 1-800-848-0829.



Animal Care Products

3M Center, Building 270-2N-03
St. Paul, MN 55144-1000